



VAH listed

# Prent Des

## Lotion for the disinfection and cleaning of castings and dental workpieces

- ready to use
- dimensional accuracy guaranteed
- does not affect surface



### Product properties

Ready to use lotion for disinfection and cleaning of castings made of polyether, silicones, alginates and hydrocol-loids and for all other dental workpieces.

### Composition

Active ingredients in 100 g lotion: Benzalkonium chloride 0,48 g; Glyoxal 0,20; Glutaraldehyde 0,01 g.

### Chemical physical data

Appearance: light blue, clear lotion  
 pH-value: 5,5 +/- 1,0  
 Density (at 20 °C): 1,000 +/- 0,003

### Areas of Application

Ready to use lotion for rapid disinfection and thorough cleaning of dental workpieces and dentures like castings made of polyether, A/C silicones, alginates and hydro-colloids.

### Direction for use

Bathe castings made of alginates and hydrocolloids in lotion for a short period and let them dry for 5 minutes afterwards. Bathe other materials in lotion for 5 minutes and rinse with water afterwards.

### Application time

Bactericidal incl. MRSA, yeasticidal acc. to VAH-guidelines tested at high bioburden level	5 min.
HBV/HIV/HCV, BVDV, Vaccinia virus (limited virucidal effectiveness acc. to RKI/DVV)	1 min.

### Delivery Forms

Single package 5 litres container	Sales unit 1 x 5 L	Ref.-No. 09983
--------------------------------------	-----------------------	-------------------

### Pharmaceutical expert reports

**Prof. Dr. Schubert, Frankfurt/Main:**  
 Certification for disinfection of surgical instruments in accordance with DGHM/VAH norm from 15.04.2010

**Dr. Suchomel, Wien:**  
 Certification for disinfection of surgical instruments in accordance with DGHM/VAH norm from 25.05.2010


### Special information

Contains glyoxal. Can cause allergic reactions. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Dispose of contents/container to approved disposal company or local collection. Keep out of the reach of children.

Use disinfectants in a safe way. Always read label and product information before use.



AMPri Handelsgesellschaft mbH  
 Benzstr. 16 / 21423 Winsen (Luhe), Germany  
 Tel. +49 (0)4171 - 8480-0 / www.ampri.de / info@ampri.de

 bioSight GmbH  
 Brasselsbergstr. 3 / 34132 Kassel, Germany  
 Tel.: +49 (0)561 - 81678111



PZN - 9668018