

Technical Data Sheet

Article-No.: **09801**
 Description: **MED-COMFORT**
Digital Thermometer



DIMENSIONS*					
length	width	height	diameter	capacity	weight
128 mm	18 mm	-	-	-	-

PRODUCT DESCRIPTION							
material	☐ Polypropylene (PP)	☐ Polyethylene (PE)	☐ PP non woven	☐ PP non woven with PE coating	☐ paper	☒ PVC & steel	
colour	☐ blue [B]	☐ cedro [C]	☐ dark blue [DB]	☐ green [G]	☐ green opac [GO]	☐ purple [L]	☐ lightblue [LB]
	☐ Lime [LIME]	☐ Mix [MIX] black, cedro, purple, pink, lightblue	☐ orange [O]	☐ pink [P]	☐ red [R]	☐ rose [ROSE]	☐ black [SCH]
	☐ transparent [T]	☐ violett [V]	☐ white [W]	☐ wine red [WR]	☐ yellow [Y]	☐ silver	☒ white
characteristics	☐ single use	☐ single bladed	☐ 5 ply	☐ lint free	☐ 9 ply	☒ measurement accuracy $\pm 0,1^{\circ}\text{C}$ waterproof	
ingredients	☐ 50% Urea (CAS 57-13-6)	☐ 50% water	☐ 70 % isopropyl alcohol	☒ latex-free			
remark	Not suitable for small children. Measuring range 32°C to 43°C.						

REGULATORY AFFAIRS							
PPE-Regulation (EU) 2016/425	☐ Category I	☐ Category II	☐ Category III	☒ no PPE-article			CE
MD-Regulation (EU) 2017/745	☐ Class I	☒ Class II	☐ Class III	☐ sterile	☐ measuring function	☐ no medical device	
Food Contact (EG) 1935/2004	☐ acidic foods	☐ aqueous foods	☐ fatty foods	☐ alcoholic foods	☐ dry foods	☒ not approved for food-contact	
Cosmetic Product (EG) 1223/2009	☐ cosmetic product	☒ no cosmetic product					

Technical Data Sheet

Article-No.: **09801**
Description: **MED-COMFORT**
Digital Thermometer



LOGISTIC DATA SUBPACKING	
general information	
material	carton/PVC
pieces per subpacking	1
GTIN subpacking blue [B]	-
GTIN subpacking cedro [C]	-
GTIN subpacking dark blue [DB]	-
GTIN subpacking green [G]	-
GTIN subpacking green opac [GO]	-
GTIN subpacking purple [L]	-
GTIN subpacking lightblue [LB]	-
GTIN subpacking Lime [LIME]	-
GTIN subpacking Mix [MIX]	-
GTIN subpacking orange [O]	-
GTIN subpacking pink [P]	-
GTIN subpacking red [R]	-
GTIN subpacking rose [ROSE]	-
GTIN subpacking black [SCH]	-
GTIN subpacking transparent [T]	-
GTIN subpacking violet [V]	-
GTIN subpacking white [W]	-
GTIN subpacking wine red [WR]	-
GTIN subpacking yellow [Y]	-
GTIN subpacking silver	-
GTIN subpacking	4044941040066
measures & size	
length	135 mm
width	35 mm
height	16 mm
weights	
size	gross weight*
uni	31 g

LOGISTIC DATA PALETTE	
general information	
kind of palette	euro-palette
measures & size	
cartons per layer	6
layer per palette	6
height of the palette	156 cm
weights	
size	brut weight*
uni	378 kg

LOGISTIC DATA outerR PACKING	
general information	
material	carton
subpackings per outerr packing	300
GTIN outer packing blue [B]	-
GTIN outer packing cedro [C]	-
GTIN outer packing dark blue [DB]	-
GTIN outer packing green [G]	-
GTIN outer packing green opac [GO]	-
GTIN outer packing purple [L]	-
GTIN outer packing lightblue [LB]	-
GTIN outer packing Lime [LIME]	-
GTIN outer packing Mix [MIX]	-
GTIN outer packing orange [O]	-
GTIN outer packing pink [P]	-
GTIN outer packing red [R]	-
GTIN outer packing rose [ROSE]	-
GTIN outer packing black [SCH]	-
GTIN outer packing transparent [T]	-
GTIN outer packing violet [V]	-
GTIN outer packing white [W]	-
GTIN outer packing wine red [WR]	-
GTIN outer packing yellow [Y]	-
GTIN outer packing silver	-
GTIN outer packing	4044941040073
measures & size	
length	470 mm
width	300 mm
height	235 mm
weights	
size	gross weight*
uni	9.800 g

Technical Data Sheet

Article-No.: **09801**
Description: **MED-COMFORT**
Digital Thermometer



WARNINGS AND SAFETY INFORMATION

storage	Store items in original packaging in a cool and dry place without additional weight, protect from direct sunlight.
use and control	1. Clean with disinfectant wipe before use. 2. Gently press the power button on the thermometer (do not use fingernails). 3. Short signal informs when the thermometer is ready for use. 4. The last measured temperature is displayed, if the button is pressed for 2 seconds. 5. Can be used orally (in the mouth), rectally (in the back) or under the arm. 6. The measurement is terminated when the signal sounds. 7. It is recommended to turn off the unit. The device will automatically turn off after 10 minutes.
cleaning instructions	Clean with wipe disinfection before and after use. Do not use aggressive cleaning agents or disinfectants.
disposal	Dispose of the product without batteries in accordance with the disposal regulations of your local waste disposal company. Used batteries must not be disposed of with household waste. Consumers are obliged to take batteries to a suitable collection point at a retailer or local authority. You can also return your used batteries to us. Used batteries may contain harmful substances or heavy metals that can damage the environment and health. Batteries are recycled; they contain important raw materials such as iron, zinc, manganese and nickel.
warnings / safety information	The thermometer is used exclusively for measuring the human body temperature, and should not be left with unattended children (risk of swallowing small parts). Do not bend thermometer, and make sure to protect it from sunlight and heat. Avoid contact with chemicals. Do not use aggressive cleaning agents and disinfectants. Only open to change battery.

*slight deviations possible due to standard tolerances

rev-no.: 2025-00
date: 22. 10.2025

changes and errors excepted