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Technical specifications / Specifiche tecniche

Caractéristiques techniques / Tekniske data / Technische gegevens

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User manual / Istruzioni per l'uso /
Notice d'utilisation / Brugsanvisning / Gebruiksaanwijzing

DE/EN

Art.-Nr. / Item No. 47 160-162 Steckbecken, Kunststoff / Bedpan, plastic			
Material:	Polypropylen / Polypropylen	Höhe / Height:	ca. / approx. 10 cm
Durchmesser / Diameter:	ca. / approx. 31 cm	Fassungsvermögen / Capacity:	ca. / approx. 2,2 l
Länge / Length:	ca. / approx. 43 cm	max. Belastung / max. Load:	120 kg
Lebensdauer / Lifetime:	ca 2 Jahre / approx. 2 years	Farbe / Color:	47 160: elfenbein / ivory 47 161: grün / green 47 162: blau / blue

IT/FR

Art.-N. / N° d'art. 47 160-162 Padella, plastica / Bassin de lit, plastique			
Materiale / Matériau:	Polipropilene / Polypropylène	Altezza / Hauteur:	ca. / env. 10 cm
Diametro / Diamètre:	ca. / env. 31 cm	Capacità massima / Contenance:	ca. / env. 2,2 l
Lunghezza / Longueur:	ca. / env. 43 cm	max. Carico / Charge max.:	120 kg
Durata media / Durée de vie:	ca. 2 anni / env. 2 ans	Colore / Couleur:	47 160: avorio / ivoire 47 161: verde / vert 47 162: blu / bleu

DA/NL

Varenr. / Art.nr. 47 160-162 Bækken, plast / Ondersteek, kunststof			
Materiale / Materiaal:	Polypropylen / Polypropyleen	Højde / Hoogte:	ca. 10 cm
Diameter / Diameter:	ca. 31 cm	Indhold / Inhoud:	ca. 2,2 l
Længde / Lengte:	ca. 43 cm	maks. Belastning / Max. belasting:	120 kg
Produktlevetid / Levensduur:	ca. 2 år / 2 jaar	Farve / Kleur:	47 160: elfenbensfarvet / ivoorkleur 47 161: grøn / groen 47 162: blå / blauw

Steckbecken, Kunststoff

Bedpan, plastic

Padella, plastica

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Bækken, plast

Ondersteek, kunststof

Art.-Nr. / Item No. / Art.-N. / N° d'art. / Varenr. / Art.nr. 47 160:
elfenbein / ivory / avorio / ivoire / elfenbensfarvet / ivoorkleur

Art.-Nr. / Item No. / Art.-N. / N° d'art. / Varenr. / Art.nr. 47 161:
grün / green / verde / vert / grøn / groen

Art.-Nr. / Item No. / Art.-N. / N° d'art. / Varenr. / Art.nr. 47 162:
blau / blue / blu / bleu / blå / blauw

[HMV-Nr. 51.40.01.0032]

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EN User Manual

Intended use:

The bedpan (also called stool, slider, etc.) is a medical device and can simplyfy emptying of bladder and bowel in bed. It is characterized by its stable processing and ergonomic shaping, which facilitate the application to the person in need of care.

Description:

The bed-pan is characterized by its stable workmanship and ergonomic design, which facilitates the application for the patient or the carer.

Indication:

for persons with limited mobility or bedrest, who can not leave the bed for using the toilet.

Contraindication:

unstable body or vertebral injuries, insufficient physical or mental capacity to use the device.

Before first use:

First, read the instruction for use in its entirety and keep it until the end of the product life. Please check if the product is undamaged before first use and any use by visual inspection. In case of signs of damage or wear, the bed-pan must not be used (any more). Further maintenance is not required if the safety instructions are observed. The bed pan must only come into contact with intact, uninjured skin. Any use other than that mentioned above is not intended. Sundo GmbH is not liable for any resulting damages, the risk is borne exclusively by the user. Any serious incidents arising in connection with this product must be reported to the manufacturer or the competent authority without delay.

Application:

- Place a few sheets of toilet paper or provided hygiene bag in the bedpan.
- The person in need of care should lie in the bed in the supine position, since the position of the buttocks must then only be changed insignificantly. The bedpan can be pushed easily under the patient after a slight lifting of the pelvis, the handle points to the nursing person. If the bedridden patient is able to change her/his position (with help), she/he can also be turned to the side. The bedpan is then carefully pushed between bed and body, and the patient is turned over back to the bedpan (protect the patient from rolling out of the bed!).
- Check the correct position of the bedpan before you tuck in the patient and leave the room if necessary.
- After the call or ringing of the patient, refold the blanket as necessary, wipe the outer genitalia with toilet paper. Carefully remove the bedpan, place the lid and put it aside. For hygiene reasons, place the bedpan on a storage facility equipped with a one-off pad (not on the bed, bedside table or similar).
- Then perform the required body cleansing and care as usual.
- Empty, clean and disinfect the bedpan.

Cleaning and disinfecting:

- The bedpan can be cleaned with water and a mild household cleaner, please make sure that it does not contain any oils or greases.
- It can be disinfected with a commercial, chlorine-free disinfectant.
Important: When using disinfectants, the manufacturer's instructions must be observed.
Preparation in the case of inter-patient use in nursing homes, hospitals, etc.:
- The bedpan must only come into contact with intact, uninjured skin. According to the guideline of the "DGKH, DGSV, AK: Validation and Routine Monitoring of Thermal Cleaning and Disinfection Processes for medical Devices..." / ISO 15883-3 the application of an A0-value above 600 is necessary.

Safety instructions:

- Caution, choking hazard! Product and its packaging (polybag) are not toys, do not leave children unattended with it!
- The permissible body weight of max. 120 kg must not be exceeded.
- As a caregiver, take the necessary protective measures (disinfection of the hands, wearing disposable gloves, etc.)
- Disinfect the bedpan after every use!
- A bedpan can create pressure points, especially in patients who are bedridden, do not allow the patient to lie down on it longer than necessary!

Warranty:

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly. We reserve the right of modifications serving the technical improvement.

Product and packaging must be disposed of in accordance with local regulations.

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